**STERILE
CUTTERS AND BURRS
for FMS shavers**

CE 0197

Distributed By: **Depuy Mitek, Inc.**
325 Paramount Drive
Raynham, MA 02767, USA
Tel: 1-800-356-4835

FUTURE MEDICAL SYSTEMS S.A.
265, Rte de la Baronne 06610
St Jeannet, France
Tel: +33 4 92 12 04 74

● DePuy Mitek
a *Johnson & Johnson* company

LIT014M/F

**DePuy Mitek P/N: 109094
Rev A 04/19/2007**

ENGLISH

STERILE Cutters and Burrs for FMS Shavers

INSTRUCTIONS FOR USE

INDICATION FOR USE:

FMS shaver cutters and burrs are intended to provide controlled cutting, burring, shaving and abrading of bone and tissue during orthopedic procedures of the knee, shoulder, elbow, wrist, ankle, hip, small joints and Temporal Mandibular Joint (TMJ).

WARNING:

- Adequate irrigation to the tip of the blade or burr is required to cool the blade and prevent accumulation of excised material in the surgical site.

CAUTION:

- FMS disposable blades are packaged as a set. Do not interchange blade components.
- Do not bend blade.
- Do not allow blade or burr to come in contact with other instruments while rotating.
- Excessive side loading does not improve cutting and may result in increased blade wear and degradation of the blade.
- Do not use blade or burr without irrigation fluid or the blade may seize.

INSTALLATION:

- With your thumb, rotate the blade-locking ring to the left.
- Align the blade-locking pin with the handpiece locking slot, and push in.

STORAGE CONDITION:

Sterile, unless the package is damaged or open.

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