**Hand Control Interface to:**  
**Arthrex (Adapteur™ Power System II)**  
**115V 50-60Hz Ref. 282000**  
**Arthrex (Adapteur™ Power System II)**  
**230V 50-60Hz Ref. 282200**

**DePuy Mitek, Inc.**  
325 Paramount Drive  
Raynham, MA 02767, USA  
Tel: 1-800-356-4835

**FUTURE MEDICAL SYSTEMS S.A.**  
265, Rte de la Baronne 06640  
St Jeannet, France


**LIT281M/B**

**DePuy Mitek P/N: 109339**  
**Rev B 01/2008**

**Hand Control Interface to:**  
**Arthrex (Adapteur™ Power System II) 115V 50-60Hz Ref. 282000**  
**Arthrex (Adapteur™ Power System II) 230V 50-60Hz Ref. 282200**

**Hand Control Interface to: Arthrex (Adapteur™ Power System II) 115V 50-60Hz Ref. 282000**  
**Hand Control Interface to: Arthrex (Adapteur™ Power System II) 230V 50-60Hz Ref. 282200**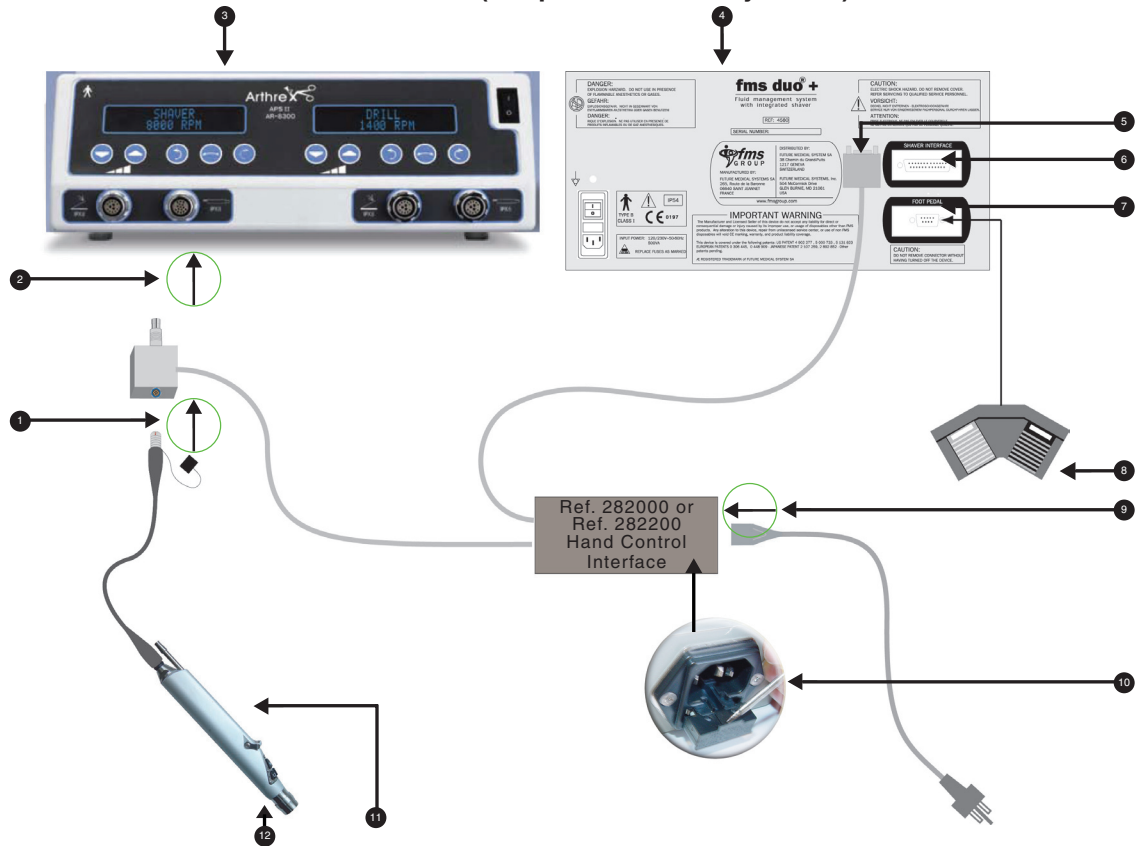

## INSTRUCTIONS FOR USE

### Hand Control Interface to: Arthrex (Adapteur™ Power System II) 115V 50-60Hz Ref. 282000

### Hand Control Interface to: Arthrex (Adapteur™ Power System II) 230V 50-60Hz Ref. 282200

The Ref. 282000 or Ref. 282200 interface allows the interfacing of the shaver, FMS DUO®+ pump, or FMS SOLO® pump. When using the following shaver hand control interface, the activation of the pump functions are achieved with a 2 - way foot pedal Ref. 284172. The activation of the shaver is achieved using the shaver manufacturer's hand control (button on shaver) or shaver foot pedal.

NOTE: Ensure you use the correct interface cable when connecting to another shaver.

### WARNINGS:

- The system may be affected by electromagnetic interference from other instruments. Verify that all other instruments and appliances (associated or not with the system) comply with the standard EN 60601-1-2 (EMC).
- If the system continues to be affected, isolate it from the offending instrument and connect to a different main output.
- Electrical safety testing should be performed by a biomedical engineer or other qualified person at least once per year.
- Risk of electrical shock: Do not remove the cover.
- Do not connect the device to a power source that is not properly earthed (grounded).
- Disconnect the device from the main power source when cleaning, servicing, or inspecting.
- Inspect equipment and cables for wear before each use. If damage is noted, contact DePuy Mitek representative to replace worn cables.
- Avoid fluid contact with the interface, and its electrical connectors.
- Do not use flammable agents when cleaning the interface.
- To avoid risk of fire, replace fuses with same type and rating.

### SYSTEM INDICATORS:

- ① + ② + ⑤ + ⑨ Connections
- ③ Arthrex (Adapteur™ Power System II)
- ④ FMS DUO+ or FMS SOLO
- ⑥ 25 pin shaver Interface connection
- ⑦ Foot pedal connection
- ⑧ Foot pedal Ref. 284172
- ⑩ Fuse holder: 2x0.1A fuses, Time delay Ref. 218.100 Littlefuse
- ⑪ Hand control activation
- ⑫ Handpiece Arthrex (Adapteur™ Power System II)

### SPECIFICATIONS:

#### Technical reference:

#### Electrical specifications

Voltage: 115V for Ref. 282000 and 230V for Ref. 282200  
 Frequency: 50-60Hz  
 Total RMS current: 6mA  
 Power: 0.9W  
 Fuses: 0.1A Time Delay  
 Conforms to standard EN 60601-1, UL 60601-1, CSA-C22.2n° 601.1 and EN 60601-1-2 (EMC)

#### Storage conditions

Room temperature: -10°C (14°F) to + 50°C (122°F)  
 Relative humidity: 30 to 70%  
 Atmospheric pressure: between 500 hPa and 1060 hPa

#### Operating conditions

Room temperature: +10°C (50°F) to + 40°C (104°F)  
 Relative humidity: 30 to 75%  
 Atmospheric pressure: between 700 hPa and 1060 hPa

#### Cleaning the Interface:

At the end of the operating day, switch the equipment off and unplug the Interface. Wipe off the Interface with a soft cloth that has been slightly moistened with a neutral pH detergent. Wipe again using distilled water to rinse. DO NOT SOAK.

DO NOT AUTOCLAVE.

Type B  
Class I

04120000

Year Week Serial number

| Troubleshooting:                           |  |
|--|--|
| Problems                                   | Possible solutions   |
| The shaver works and the pump does not run | <ul style="list-style-type: none"> <li>- Verify the connections ⑤</li> <li>- Verify the main connections ⑨</li> <li>- Verify the two fuses ⑩</li> <li>- Replace the two fuses by fuses of the same type: 0.1 Time delay, Ref. 218.100 Littlefuse</li> <li>- If the pump does not run, please contact an authorized repair center.</li> </ul> <p>In the US, call DePuy Mitek at 800-382-4682, option 2.<br/>                     In Europe, contact your local affiliate or sales representative.</p>   |
| The shaver does not run                    | <ul style="list-style-type: none"> <li>- Disconnect ① and ② reconnect straight to the Arthrex (Adapteur™ Power System II).</li> <li>- If the shaver does not work, contact the console manufacturer.</li> <li>- If the shaver works reconnect all connections, disconnect ⑤ and verify your handpiece.</li> <li>- If the handpiece does not work, return the interface to DePuy Mitek by calling 800-382-4682, option 3.</li> <li>- If the handpiece works, Contact DePuy Mitek Customer Support by calling:</li> </ul> <p>In The US, call 800-382-4682, option 2.<br/>                     In Europe, contact your local affiliate or sales representative.</p> |

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### Interfaz de control manual para: Arthrex (Adapteur™ Power System II) 115V 50-60Hz Ref. 282000

### Interfaz de control manual para: Arthrex (Adapteur™ Power System II) 230V 50-60Hz Ref. 282200

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